

WHAT IS THE EXCI308 EXCIMER SYSTEM?

Exci308 is an excimer lamp that emits a Monochromatic excimer light at 308 nm. This 308 nm wavelength is known to be very efficient and the best to treat auto-immune skin diseases with very limited side effects

HOW DOES EXCI308 COMPARED TO EXCIMER LASERS?

Altogether excimer lamp has the same clinical efficiency as an excimer laser but is much less costly as it doesn't need consumables. Afterwards, Exci308 enables fast treatment sessions thanks to its large spot size and its power density emitting shots of 100mJ/s

IS EXCI308 AN EXCIMER LASER?

Exci308 is an excimer lamp, it has the same technique as excimer laser; "Monochromatic Excimer Light" technology. Additionally, both Laser and Exci308 uses xenon-chloride gas.

WHAT IS THE BULB LIFETIME?

Clarteis has conducted extensive reliability tests on the bulb, as a result, Clarteis offers a 3 years warranty for the bulb with unlimited use.

CAN PATIENTS USE THIS DEVICE AT HOME?

No, Exci308 is only professional used and made exclusively for use by qualified dermatologists.

CAN EXCI308 BE USED ON ALL SKIN PHOTOTYPES AND ANY BODY PART?

Exci308 is suitable for all skin phototypes and can be used in most of parts of the body (excluding mucous area)

IS THIS TREATMENT SUITABLE FOR CHILDREN?

Clinical studies have demonstrated the safety of use of Monochromatic Excimer Lamps as Exci308 in the treatment of children above the age of 15 years old.

CAN EXCI308 BE COMBINED WITH OTHER TREATMENTS?

Many dermatologists around the world have successfully combined Exci308 with topic treatments. Results are sometimes better.

ARE THERE RELAPSES FOLLOWING EXCI308 TREATMENTS?

Exci308 cure the symptoms of the disease, hence Relapse can occur. It is the case of all treatment available on the market.

WHAT ARE THE SIDE EFFECTS WITH THE EXCI308 TREATMENT ?

Exci308 has a limited sides effects. Erythema and Hyperpigmentation can occur and last for couple of weeks.

WHAT ARE THE CONTRAINDICATIONS WITH EXCI308 TREATMENT ?

Skin dermatoses with Kobner phenomenon , Lupus erythematosus, Pacemakers, Hyper-photosensitivity, Melanoma and non-melanoma skin cancers, Drugs with photosensitizer side effect, Radiotherapy, Pregnancy (no study available), Children below 15 (only a limited number of studies are available for this category), Diseases that are contagious by contact.

SETTING EXPECTATIONS

The number of treatments needed varies from person to person depending on wants, needs and the degree of the condition. In order for these treatment options to be effective, patients will need to regularly attend phototherapy appointments. In some instances, patients might need to visit the office multiple times per week.

DO I NEED TO USE TOPICAL ANESTHESIA? WHAT ABOUT GELS?

No. Anesthetics and cooling gels are not needed with Exci308, because Monochromatic UVB lamp makes it uniquely pain-free. However, some buildup of heat will be experienced with smaller spot sizes. This unique benefit saves time, cost and mess by eliminating gels and sprays required by other UVB lamps that cool the skin during treatment.

EXCI308 ADDRESSES CONDITIONS FOR PATIENTS WITH:

- ✓ ATOPIC DERMATITIS
- ✓ LEUKODERMA
- ✓ PSORIASIS
- ✓ SCALP PSORIASIS
- ✓ VITILIGO